

Home Care Technical Specification


Fold Down Shower Seat

DN7110

Stock Numbers:

- DN7110 – Folding Down Shower Seat

Dimensions:

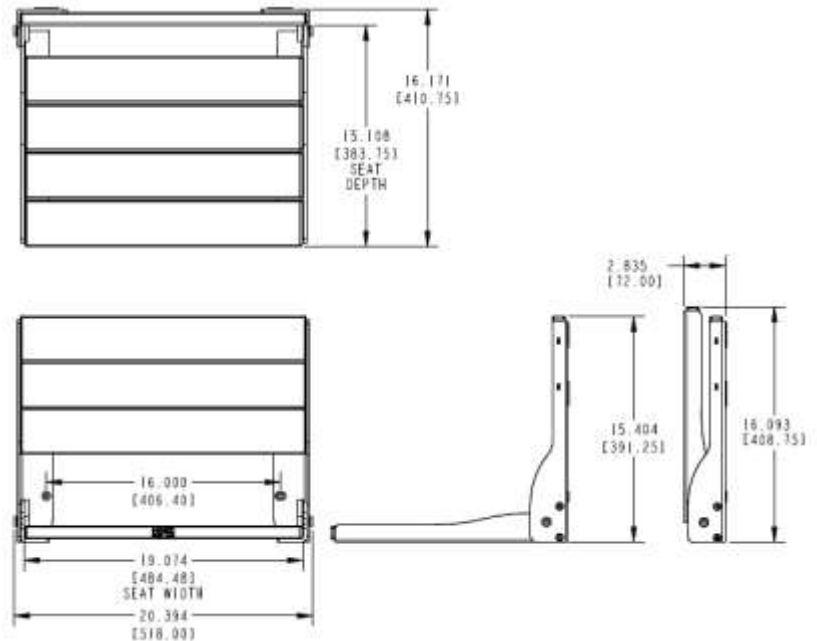

Materials:

Folding Down Shower Seat is made of Aluminum and Teak Wood

**ASSEMBLY TESTED
TO CONFIRM ADA
COMPLIANCE**



WARNING:
TO PREVENT SERIOUS OR FATAL INJURY:
Insure product is assembled and used properly.
FOR USE ONLY BY PERSONS WEIGHING
LESS THAN 400 POUNDS. Failure to follow
correct assembly procedures can cause the seat
to become unstable. DO NOT jump on, stand on
or tilt. Contact CSI immediately at 1-800-882-
0116 if any components are missing or damaged.
Do not modify or substitute any components.
FOR QUESTIONS OR ASSISTANCE WITH
ASSEMBLY CONTACT CSI AT 1-800-882-0116.
Consult healthcare professional for advice and
instructions specific to user's needs before use.

Installation Instructions:

1. Remove the top two back rest panels to reveal the top four mounting holes.
2. Identify the mounting location of the shower seat making sure the shower seat is level.
3. Mark the six mounting hold locations once the desired location is determined.
4. Set aside the shower seat and drill 1/8" pilot hole through tile in each mounting location. Drill a 1/4" hole through the tile only at the same locations to allow the wood screw to pass through.
5. When complete, ensure the wall is clear of any debris.
6. Secure the shower seat to the wall, using the provided six wood mounting screws. Start with the top two mounting hole locations.
7. Once all screws have been installed and tightened, slide the back panels into place.
8. Installation is complete – Test the seat to make sure it is secure.

Stock Number: _____ Shower Mesh Seat
By Creative Specialties



25300 AL MOEN DRIVE • NORTH OLMSTED, OH 44070
CORPORATE HEADQUARTERS: (800) 321-8809 • FAX: (800) 848-6636
INTERNATIONAL: (440) 962-2000 • FAX: (440) 962-2726
CUSTOMER SERVICE (USA): 800-882-0116 • FAX: (888) 379-2720
WWW.MOEN.COM

©Copyright Creative Specialties International

Literature #BA1593